




SUPER INDUCTIVE SYSTEM

WONTECH presents an innovative therapy - the Super Inductive System (SIS). The technology is based on high-intensity electromagnetic field, which positively influences the human tissue. Therapeutic effects include pain relief, fracture healing, myorelaxation, myostimulation and joint mobilization.

- As SIS targets the neuromuscular tissue, it can be used to treat acute and chronic pain conditions of the musculoskeletal and neural system.
- With specific settings the interaction between the electromagnetic field and the muscle leads to desirable muscle contraction. Through repetitive contractions of the muscles surrounding the joint, release of joint blockage can be achieved.
- Due to enhancement of the local blood circulation and tissue metabolism, SIS is indicated in fracture healing process from its very beginning.
- To strengthen weakened muscles, SIS causes muscle contractions by triggering the action potential in the neuromuscular tissue.
- With its high frequency of stimulation, SIS effectively eliminates muscle tension and therefore can be used in treatment of spastic disorders.
- With the 650mm laser at the center of the electromagnetic coil and combines it into a dual function therapeutic device.

PATENTED TECHNOLOGY

The WONTECH Super Inductive System features unique high-tech solutions. Ingenious design of the coil in the applicator allows for smooth therapy procedure with the device, even under demanding operational conditions, which are frequencies of up to 150 Hz and intensity of up to 5 T. One-of-a-kind is also the coil cooling system, which allows for the most effective cooling and ensures the possibility of prolonged therapy times, even when set to values causing the highest possible load on the device.

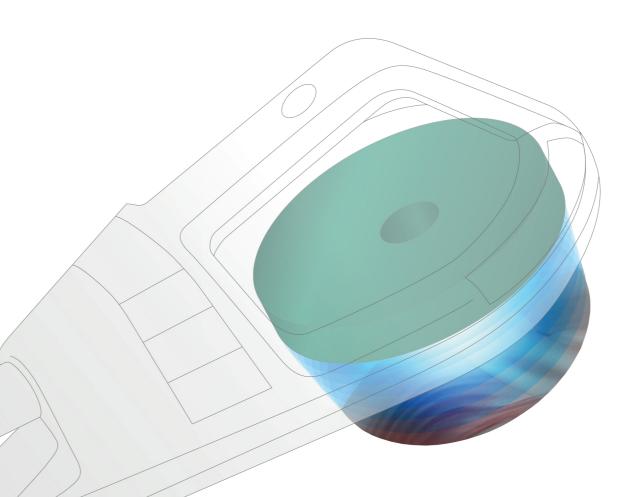

FREQUENCY-SPECIFIC PAIN MANAGEMENT


Wide range of frequencies assures that the WONTECH Super Inductive System can be indicated for all stages of painful conditions. Their management is based on three different pain control theories. Each of them varies in frequency spectrum. Therefore the SIS therapy leads to immediate relief at all stages of disorders, whether chronic or acute.

CLINICALLY PROVEN WITH EVIDENT RESULTS

PAIN REDUCTION DEMONSTRATED ON THE VISUAL ANALOGUE SCALE


IMPROVEMENT DESCRIBED BY 87 % OF THE PATIENTS



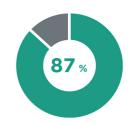

Graphics show pain reduction after 5 to 10 therapies with the WONTECH Super Inductive Inductive System. Results are demonstrated on the Visual Analogue Scale. 87 % of the participants described decrease of pain.*

*KAZALAKOVA, Krasimira: Repetitive Peripheral Magnetic Stimulation as Pain Management Solution in Musculoskeletal and Neurological Disorders - A Pilot Study

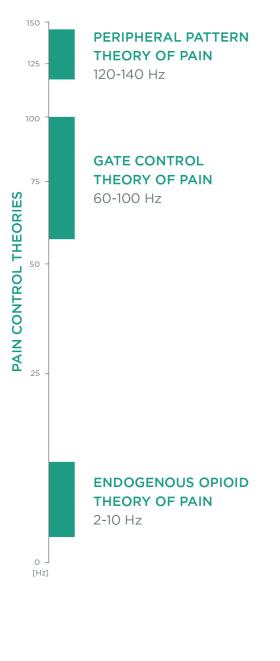

OTHER MEDICAL EFFECTS

JOINT MOBILIZATION

Joint mobilization is achieved through repetitive contractions of the muscles surrounding the joint capsule. This repetitive contraction substitutes manual joint mobilization, which leads to joint play restoration.



FRACTURE HEALING

The high-intensity electromagnetic field enhances blood circulation in the affected area and supports formation of the vascular and cartilage callus. Consequently progressive cartilage mineralization and bone remodeling are initiated.



MYOSTIMULATION


Interaction of the electromagnetic field within neuromuscular tissue, results in nerve depolarization and muscle contractions. Based on the selected stimulation frequency, muscle facilitation or strengthening can be achieved.



SPASTICITY REDUCTION

Inhibition of an increased muscle tone is achieved through affecting the spinal level of muscle tone control. This mechanism is indicated for treatment of central motor impairment, in which spasticity occurs.



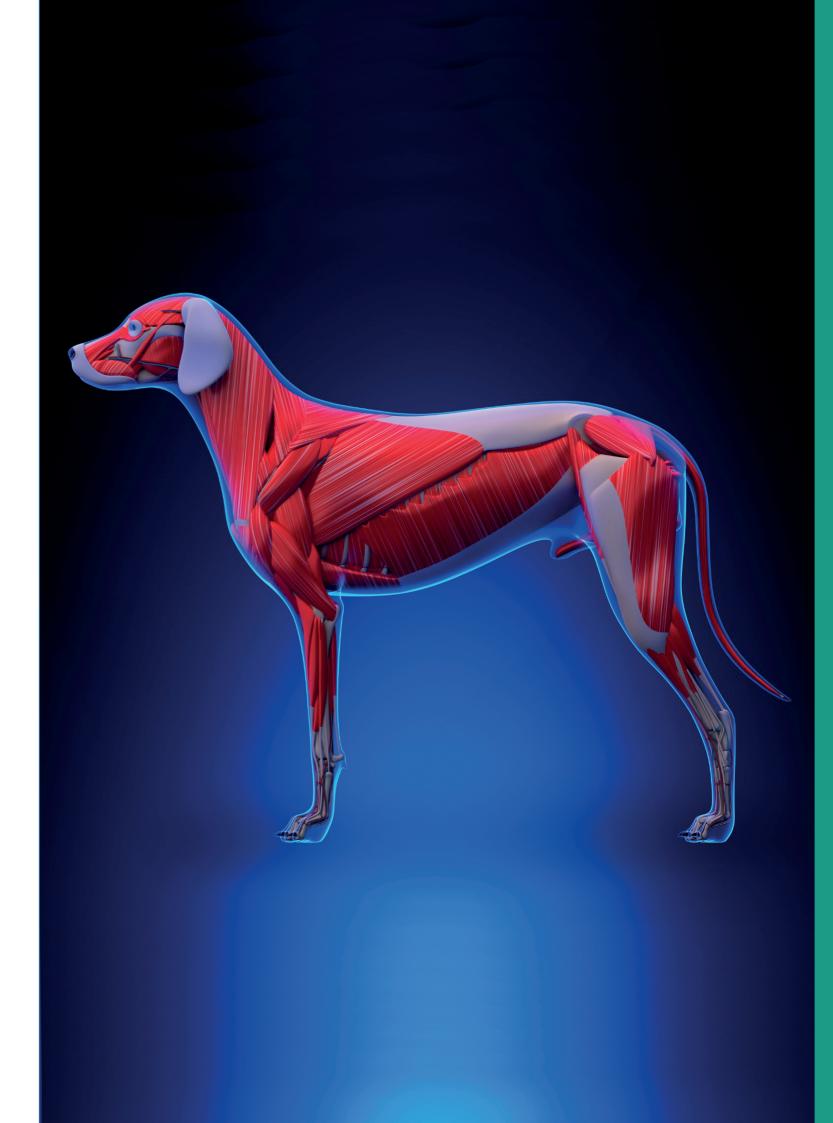



SUPER INDUCTIVE SYSTEM

WONTECH SUPER INDUCTIVE SYSTEM ELITE

- Frequency up to 150 Hz
- Intensity up to 5 Tesla
- Patented applicator technology
- 14,4 " colour touch screen
- Power up to 2000 W

FEATURES & BENEFITS

- QUICK protocols
- Body Parts navigation
- Preset protocols
- Variable five-joint arm
- Pulse quality monitor
- Trolley
















Five-joint arm
Easy applicator
positioning

THE APPLICATOR

ELECTROMAGNETIC FIELD DISTRIBUTION

The electromagnetic field emitted by the Super Inductive System applicator ensures precise therapy targeting into areas localized deep within the tissue, while delivering intensities of up to 5 T.

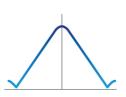

Visualization of the electromagnetic field distribution on the applicator surface



Laser 650 mm



WOUND HEALING
PRECISE TARGETING
PROMOTE BLOOD
IN-DEPTH REACH
ANTI-INFLAMMATORY
ANTI-EDEMA

TECHNICAL SPECIFICATIONS OF THE WONTECH-6000 SUPER INDUCTIVE SYSTEM

Model		
	SUPER INDUCTIVE SYSTEM(SIS)	
Part number		
User interface	10.4" colour touch screen	
QUICK protocols	•	
Body Parts navigation	•	
Preset protocols	•	
Laser wavelenght	650 μm	
Power	2000 W	
Intensity	up to 5 T	
Frequency	up to 150 Hz	
Dimensions	540x970x430 mm	
Weight	58 kg	
Mains supply	100-240 V AC, 50-60 Hz	
Standard accesories	Six-joint arm, applicator	